

NG Shield™ Quick Reference

1 Remove actuator from package and place it at the distal end of the NG tube. Do not remove the adhesive tape strips at this stage.





2 Clip the actuator closed on the NG tube and then connect the actuator's cable with the driver's cable.





3 Turn the driver ON by pressing the ON/OFF button located on the front side of the driver. Insert the NG tube through patient nose while the NG Shield system is active (tube vibration can be felt).




4 Slide the actuator on the tube to tune to optimal position (according to patient's comfort level). Apply a fixation NG strip as commonly used with NG tubes.

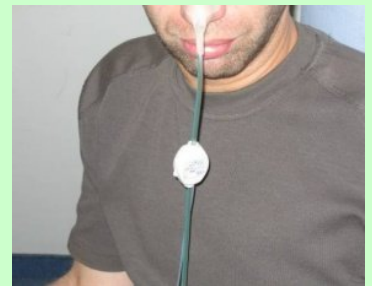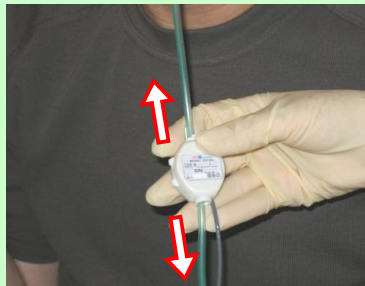

5 Open the actuator case, remove the adhesive tapes (both sides 1,2) and fix the actuator at the designated position.

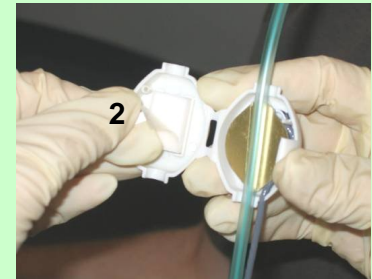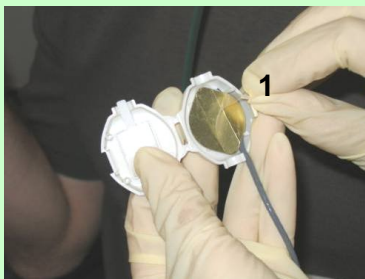

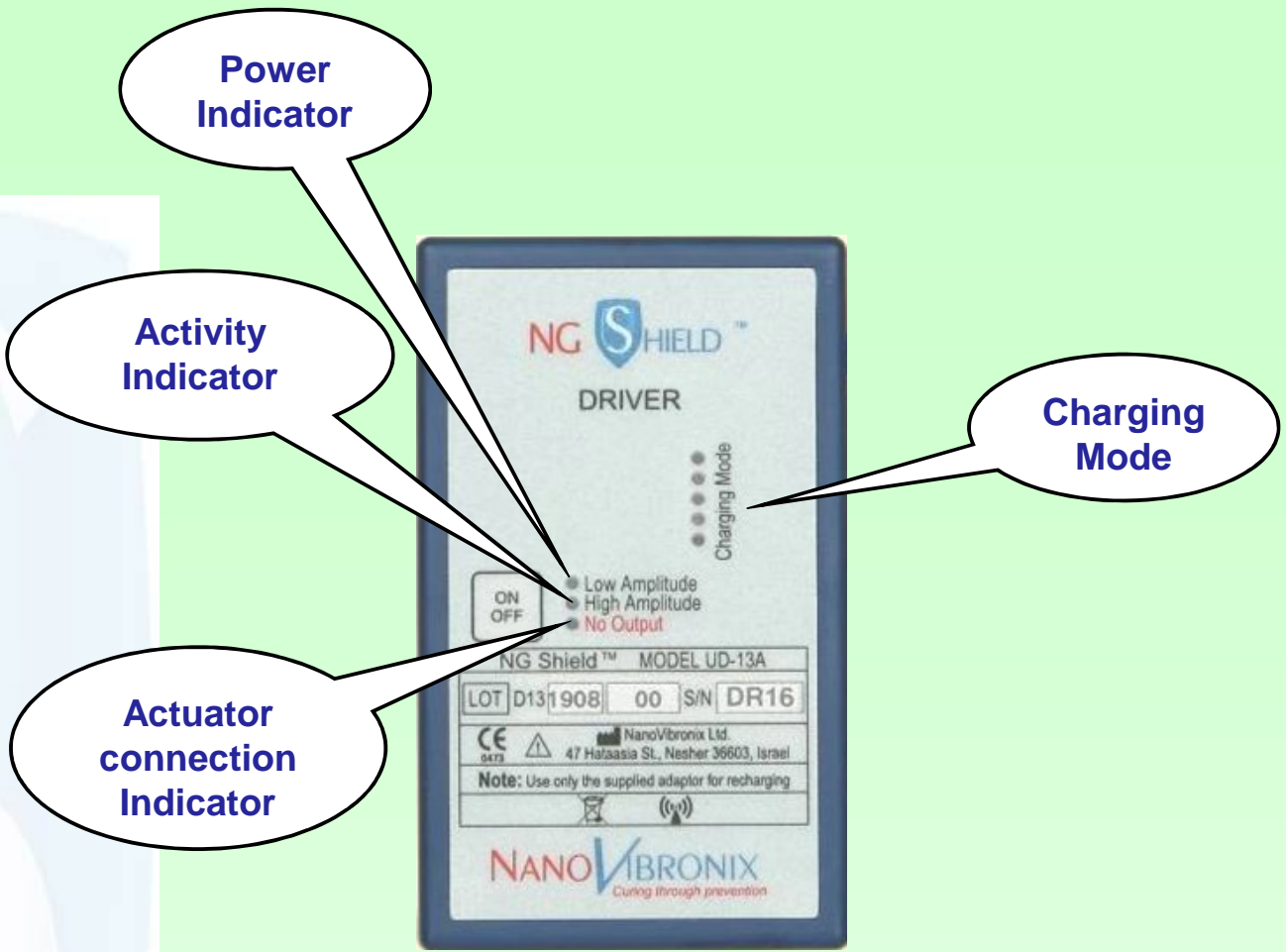

Troubleshooting

| Problem | Required action |
|--|--|
| The lower LED at the “Charging Mode” bar is blinking and audible alarm is on | Low battery – charging required Replace driver with another charged driver |
| “No Output” indicator LED (Red) is on and audible alarm is on | Actuator disconnected – check that the actuator cable is connect correctly to the driver cable |
| None of the LEDs are lit | Driver is in OFF mode. - Turn the Driver on - In case the Driver does not turn on, charge the Driver |