
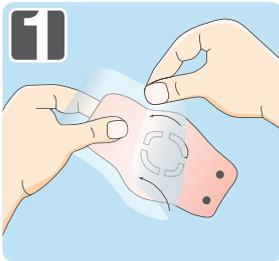


# PAINSHIELD QUICK GUIDE


Please read the complete PainShield manual before using the device and pay attention to important warnings.

For maximal effectiveness ensure that the active element is in full contact with clean, hairless and oil-free skin


For extended patch usage follow handling instructions on the patch label

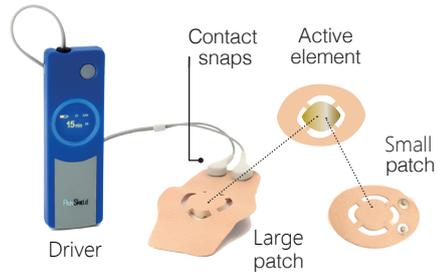

Before applying the patch, the skin should be thoroughly cleaned. Remove the protective liner from the treatment patch, and keep it for storing after use.

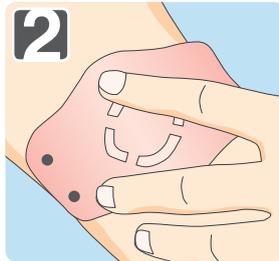

Attach the treatment patch to the skin with the adhesive side down over the area where the pain is most intense.

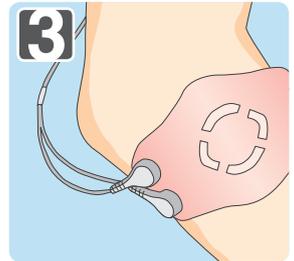

Connect the patch to the device cable using the contact snaps.

**4** Press the ON/OFF button on the device for about 2 seconds until beeping sound is heard. The device requires a few seconds to initialize and then displays the information screen.

- When the device is in working or charging modes, it enters the screen saving mode.
- To check the status press shortly on the ON/OFF button.
- The device can be turned off at any time by pressing and holding the ON/OFF button for 2 seconds.




For online guidance, please see:  
<https://youtu.be/h5NHm6apQbs>

[www.nanovibronix.com](http://www.nanovibronix.com)

**NANO VIBRONIX**<sup>®</sup>  
Therapeutic Waves in Motion

# PAINSHIELD QUICK GUIDE

Working mode  
(Active period)

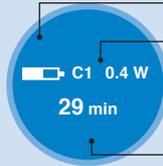

Battery level

Treatment cycle  
number: C1- C6  
0.4 W - Acoustic power


Time of treatment in the  
cycle: 0-30 min status

Working mode  
(Idle period)




Each cycle has an active period  
of 30 minutes followed by an  
inactive (idle) period of 30  
minutes. The PainShield  
battery allows for six  
treatment cycles and then  
shuts down automatically.

Treatment patch is  
disconnected or damaged

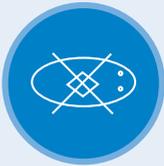

Screensaver screens:


1. In Charging mode

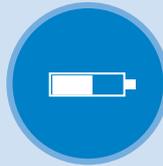

2. In Working mode



Battery is empty and  
requires charging

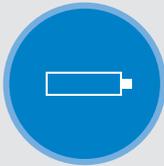

Charging in progress